

NO/DK Brukertips for "Animal" produkter
 SE Användningstips för "Animal" produkter
 FI Käytännön vinkkejä "Animal" tuotteihin
 UK User tips for "Animal" products
 DE Anwendungstipps für "Animal" Produkte

IT Guida all'uso della linea "Animal"
 FR Conseils d'utilisation pour les produits "Animal"
 NL Gebruikerstips voor "Animal" producten
 ES Consejos de aplicación para productos "Animal"

SMALL ANIMAL

BANDAGING IDEAS


Explanation:

The numbers 1-4 indicate which bandage product that is used in the illustrations


- ① AnimalSOFT
- ② AnimalVET
- ③ AnimalPOLSTER
- ④ ELASTOQUICK Sport

EAR BANDAGING


1. Shape a length of AnimalVet into a cone to protect the meatus. Make a pad to immobilize the ear.

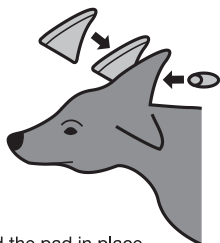

2. Hold the pad in place with AnimalVet. The discharge is absorbed and stored in the inner layers.

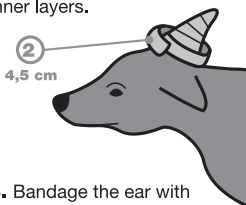


3. Bandage the ear with AnimalVet. Hold in place with a wider AnimalPolster bandage which sticks even on skin and fur. Leave an opening for the ear canal and leave the healthy ear free of bandage.

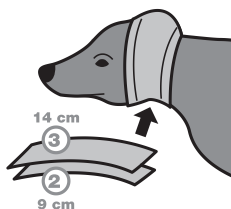


LEG BANDAGING ① ②


1. Start by wrapping one length of AnimalSoft or AnimalVet around the leg.

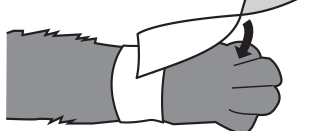

2. Tear off a suitable length to fit between the claws. Fix it to the wrap, turn it 180 degrees, slip the narrow part between the claws and fix it to the wrap on the other side. Repeat the procedure for each claw.

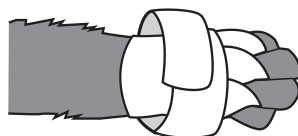



3. Complete the bandaging by a second wrapping around the leg to secure the bandages.

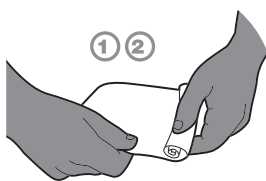

To make a pressure bandage, simply roll up AnimalSoft or AnimalVet. Follow this principle for suppurating sores to increase absorbency.

FRACTURES

1. Follow the same method used in bandaging paws. Don't forget to pad out by applying further layers of AnimalVet (see point 5).

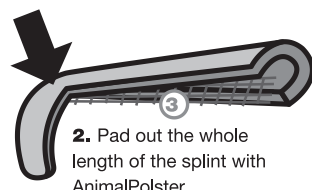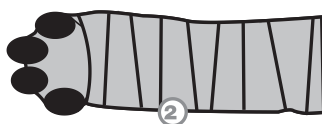


2. Pad out the whole length of the splint with AnimalPolster.

3. Hold the splint in place with ElastoQuick Sport. This method eliminates the risk of sores caused by rubbing. By allowing the claws to move freely, greater mobility is ensured without restricting the blood supply. Stimulates blood flow.



PAW BANDAGING ① ②

1. Begin by placing a length of AnimalSoft or AnimalVet between the digits and the pad.


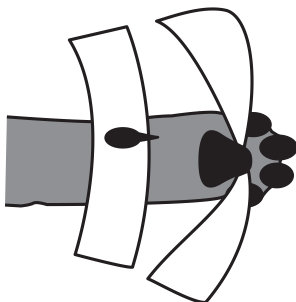


3. Cut a strip of AnimalVet to length. Make slots to go over the digits.



4. Slip the digits through the slots and fold the bandage as shown.

2. Make a slot in the AnimalVet and place a length of padding around the dewclaw.

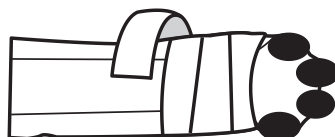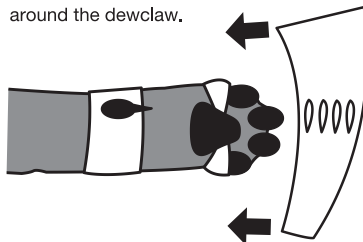


5. If necessary, wrap an additional length of AnimalVet around the paw.

LARGE ANIMAL


BANDAGING IDEAS

Explanation:
The numbers 1-3 indicate which bandage product that is used in the illustrations

- ① AnimalSOFT
- ② AnimalVET
- ③ AnimalPOLSTER

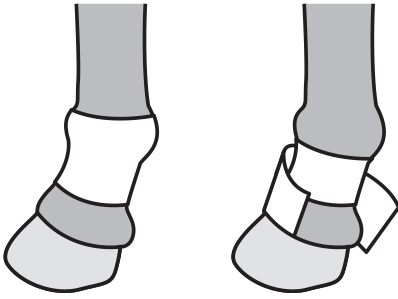


TEAT BANDAGING

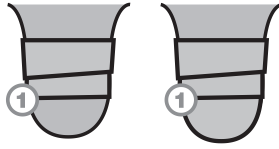

Use AnimalSoft in the event of injury to udders or fixation of teat plugs.

ELASTICITY

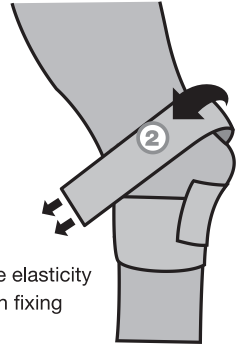




Notice how the elasticity is utilised when fixing the hamstring.

JOINT INJECTION

The bandage is placed directly onto the injection area. It is important to utilise the elasticity of the bandage to prevent it from slipping.

HOOF JOINT

When bandaging a hoof joint, the first bandage is wrapped from the pastern downward. The second bandage is placed over the injection area, and must be wrapped some way down the hoof.

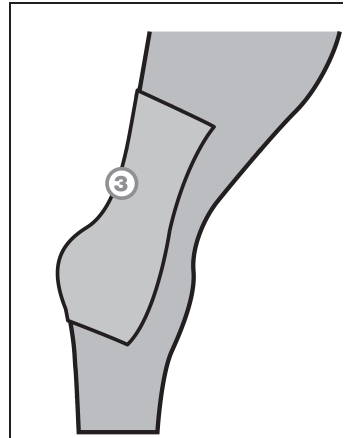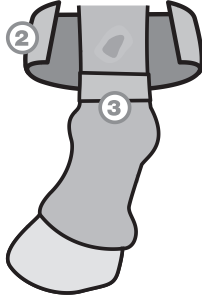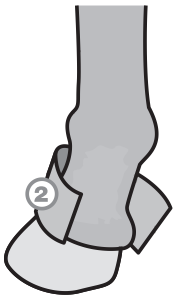

MOULD TREATMENT


One round of 14 cm AnimalVet is sufficient for mould treatment.

BANDAGING WOUNDS


Begin by wrapping one round of AnimalPolster below the wounded area. Then wrap AnimalVet around the wound to overlap the AnimalPolster by 2-3 cm. AnimalVet is replaced when required and the AnimalPolster is removed when treatment is finished.

Attach AnimalPolster directly on the area that needs protection.


Attach AnimalPolster directly on the area that needs protection, but NOT directly on the wound.

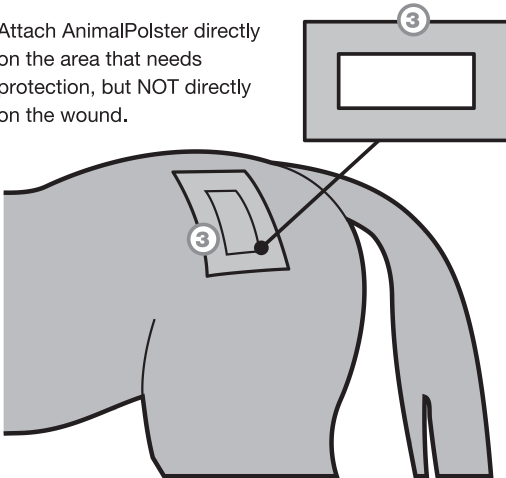

If the bandage is e.g. too loose, it is easy to tighten - see drawing.

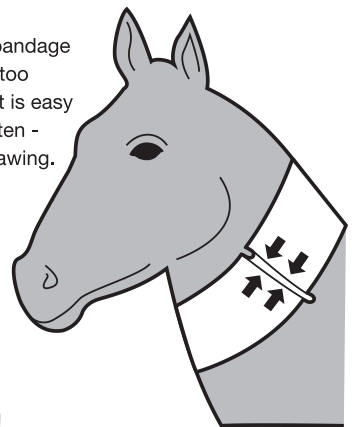

If a horse is troubled with mouth sores, the bit can be bandaged or covered with AnimalSoft to relieve pain.

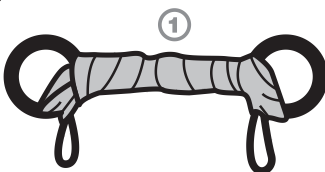

- ② ③

KNEE INJECTION

Begin by wrapping AnimalPolster under the knee. Then wrap 9 or 14 cm AnimalVet over the injection point overlapping the AnimalPolster.

This way AnimalVet will not slip downwards. For extra hold, secure the AnimalVet with another layer AnimalPolster over the knee overlapping AnimalVet.

PRINTING COLOURS PRIMA

							
BLACK							

PRINTING COLOURS SEKUNDA

							
BLACK							

PROOF-SHEET

CUSTOMER Snøgg AS	BOOKLET/LEAFLET DIMENSIONS 146x300 mm
GENERIC NAME Brukerveiledning Animal	PROOF VERSION 1
	DATE 2018-02-06
PRODUCT REF. NO/ITEM NO 209995-01	OPERATOR CLA
	APPROVED/SIGNED
CODE N/A	



First class packaging

FRONT BACK

